

For In Vitro Diagnostic Use and For Professional Use Only

Customer and Technical Service: 800-822-2947

CLIA Waived: Use lithium heparin whole blood, only
Moderate Complexity: Use lithium heparin whole blood
lithium heparin plasma, or serum

December 2009

PN: 400-7083 Rev: K

© 1997, Abaxis, Inc., Union City, CA 94587

1. Intended Use

The Piccolo[®] General Chemistry 6 reagent disc, used with the Piccolo Blood Chemistry Analyzer or the Piccolo xpress[™] Chemistry Analyzer, is intended to be used for the *in vitro* quantitative determination of alanine aminotransferase (ALT), aspartate aminotransferase (AST), creatinine, gamma glutamyltransferase (GGT), glucose, and blood urea nitrogen (BUN) in heparinized whole blood, heparinized plasma, or serum.

The tests on this panel are waived under CLIA '88 regulations. If a laboratory modifies the test system instructions, then the tests are considered high complexity and subject to all CLIA requirements. For CLIA waived labs, only lithium heparin whole blood may be tested. For use in moderate complexity labs, lithium heparinized whole blood, lithium heparinized plasma, or serum may be used.

A CLIA Certificate of Waiver is needed to perform CLIA waived testing. A Certificate of Waiver can be obtained from the Centers for Medicare & Medicaid Services (CMS). Please contact the Commission on Laboratory Accreditation (COLA) at 1-800-981-9883 for assistance in obtaining one.

2. Summary and Explanation of Tests

The Piccolo Gen Chem 13 Reagent Disc and the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer comprise an *in vitro* diagnostic system that aids the physician in diagnosing the following disorders:

Alanine Aminotransferase (ALT):	Liver diseases, including viral hepatitis and cirrhosis.
Aspartate aminotransferase (AST):	Liver disease including hepatitis and viral jaundice, shock.
Creatinine:	Renal disease and monitoring of renal dialysis.
Gamma glutamyltransferase (GGT):	Liver diseases, including alcoholic cirrhosis and primary and secondary liver tumors.
Glucose:	Carbohydrate metabolism disorders, including adult and juvenile diabetes mellitus and hypoglycemia.
Blood urea nitrogen (BUN):	Renal and metabolic diseases.


As with any diagnostic test procedure, all other test procedures including the clinical status of the patient, should be considered prior to final diagnosis.

3. Test Principles

Alanine Aminotransferase (ALT)

Alanine aminotransferase (ALT) has been measured by three methods. Two of these methods—the colorimetric dinitrophenylhydrazine coupling technique^{1,2} and the fluorescent enzymatic assay—are rarely used.³ An enzymatic method based on the work of Wróblewski and LaDue⁴ is the most common technique for determining ALT concentrations in serum. A modified Wróblewski and LaDue procedure has been proposed as the recommended procedure of the International Federation of Clinical Chemistry (IFCC).⁵

The method developed for use on the Piccolo Analyzers is a modification of the IFCC-recommended procedure. In this reaction, ALT catalyzes the transfer of an amino group from L-alanine to α -ketoglutarate to form L-glutamate and pyruvate. Lactate dehydrogenase catalyzes the conversion of pyruvate to lactate. Concomitantly, NADH is oxidized to NAD⁺, as illustrated in the following reaction scheme.

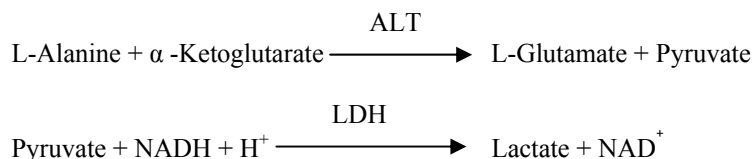


The rate of change of the absorbance difference between 340 nm and 405 nm is due to the conversion of NADH to NAD⁺ and is directly proportional to the amount of ALT present in the sample.

Aspartate Aminotransferase (AST)

The aspartate aminotransferase (AST) test is based on the Karmen rate method⁶ as modified by Bergmeyer.⁷ The current International Federation of Clinical Chemistry (IFCC) reference method utilizes the Karmen/Bergmeyer technique of coupling malate dehydrogenase (MDH) and reduced nicotinamide dinucleotide (NADH) in the detection of AST in serum.^{7,8} Lactate dehydrogenase (LDH) is added to the reaction to decrease interference caused by endogenous pyruvate.

AST catalyzes the reaction of L-aspartate and α -ketoglutarate into oxaloacetate and L-glutamate. Oxaloacetate is converted to malate and NADH is oxidized to NAD⁺ by the catalyst MDH.

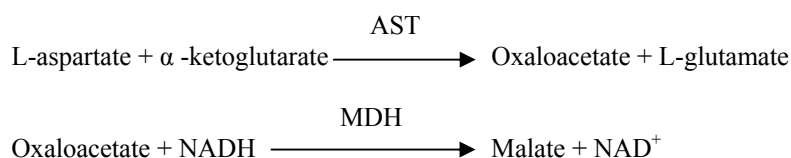


The rate of absorbance change at 340 nm/405 nm caused by the conversion of NADH to NAD⁺ is directly proportional to the amount of AST present in the sample.

Creatinine (CRE)

The Jaffe method, first introduced in 1886, is still a commonly used method of determining creatinine levels in blood. The current reference method combines the use of fuller's earth (floridin) with the Jaffe technique to increase the specificity of the reaction.^{9,10} Enzymatic methods have been developed that are more specific for creatinine than the various modifications of the Jaffe technique.^{11,12,13} Methods using the enzyme creatinine amidohydrolase eliminate the problem of ammonium ion interference found in techniques using creatinine iminohydrolase.¹⁴

In the coupled enzyme reactions, creatinine amidohydrolase hydrolyzes creatinine to creatine. A second enzyme, creatine amidinohydrolase, catalyzes the formation of sarcosine from creatine. Sarcosine oxidase causes the oxidation of sarcosine to glycine, formaldehyde and hydrogen peroxide (H₂O₂). In a Trinder finish, peroxidase catalyzes the reaction among the hydrogen peroxide, 2,4,6-tribromo-3-hydroxybenzoic acid (TBHBA) and 4-aminoantipyrine (4-AAAP) into a red quinoneimine dye. Potassium ferrocyanide and ascorbate oxidase are added to the reaction mixture to minimize the potential interference of bilirubin and ascorbic acid, respectively.

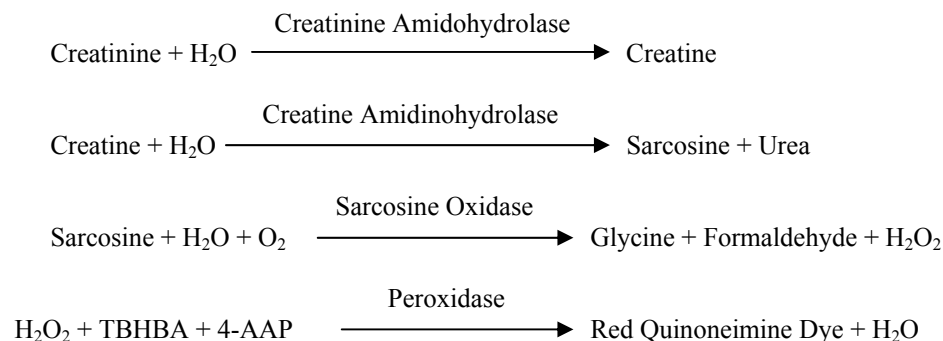

Two cuvettes are used to determine the concentration of creatinine in the sample. Endogenous creatine is measured in the blank cuvette, which is subtracted from the combined endogenous creatine and the creatine formed from the enzyme reactions in the test cuvette. Once the endogenous creatine is eliminated from the calculations, the concentration of creatinine is proportional to the intensity of the red color produced. The endpoint reaction is measured as the difference in absorbance between 550 nm and 630 nm.

eGFR (calculated)

Serum creatinine is routinely measured as an indicator of renal function. Because creatinine is influenced by age, gender and race, chronic kidney disease (CKD) may not be detected using serum creatinine alone. Thus, the National Kidney Disease Education Program strongly recommends that laboratories routinely report an estimated Glomerular Filtration Rate (eGFR) when serum creatinine is measured for patients 18 and older. Routinely reporting the eGFR with all serum creatinine determinations allows laboratories to help identify individuals with reduced kidney function and help facilitate the detection of CKD. Calculated eGFR values of <60 ml/min are generally associated with increased risk of adverse outcomes of CKD.


Calculation of the eGFR is performed by the Piccolo using the patient's age, gender and race. The Piccolo method for creatinine is traceable to the IDMS reference method for creatinine so that the following form of the MDRD equation for calculating the eGFR can be used.

$$\text{GFR (mL/min/1.73 m}^2\text{)} = 175 \times (\text{S}_{\text{cr}})^{-1.154} \times (\text{Age})^{-0.203} \times (0.742 \text{ if female}) \times (1.212 \text{ if African American})$$

Gamma Glutamyltransferase (GGT)

The first quantitative methods developed to measure gamma glutamyltransferase (GGT) involved a second reaction to form an azo dye that combined with a chromophore.^{15,16} The change to L- γ -glutamyl-*p*-nitroanilide as the substrate in the reaction eliminated the dye-formation step.¹⁷ Due to the poor solubility and stability of L- γ -glutamyl-*p*-nitroanilide, this procedure was modified to use the substrate L- γ -glutamyl-3-carboxy-4-nitroanilide.¹⁸ The International Federation of Clinical Chemistry (IFCC) recommended GGT method is based on the latter substrate, with glycylglycine as the other substrate.¹⁹

Abaxis has modified the IFCC method to react at 37°C. The addition of sample containing gamma glutamyl transferase to the substrates L- γ -glutamyl-3-carboxy-4-nitroanilide and glycylglycine (gly-gly) causes the formation of L- γ -glutamyl-glycylglycine (glu-gly-gly) and 3-carboxy-4-nitroaniline.

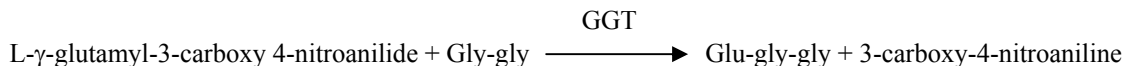


The absorbance of this rate reaction is measured at 405 nm. The production of 3-carboxy-4-nitroaniline is directly proportional to the GGT activity in the sample.

Glucose (GLU)

Measurements of glucose concentration were first performed using copper-reduction methods (such as Folin-Wu²⁰ and Somogyi-Nelson^{21,22}). The lack of specificity in copper-reduction techniques led to the development of quantitative procedures using the enzymes hexokinase and glucose oxidase. The glucose test incorporated into the Piccolo General Chemistry 6 reagent disc is a modified version of the hexokinase method, which has been proposed as the basis of the glucose reference method.²³

The reaction of glucose with adenosine triphosphate (ATP), catalyzed by hexokinase (HK), produces glucose-6-phosphate (G-6-P) and adenosine diphosphate (ADP). Glucose-6-phosphate dehydrogenase (G-6-PDH) catalyzes the reaction of G-6-P into 6-phosphogluconate and the reduction of nicotinamide adenine dinucleotide (NAD⁺) to NADH.

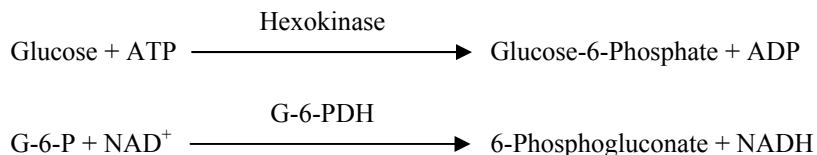


The absorbance is measured bichromatically at 340 nm and 850 nm. The production of NADH is directly proportional to the amount of glucose present in the sample.

Blood Urea Nitrogen (BUN)

Urea can be measured both directly and indirectly. The diacetyl monoxime reaction, the only direct method to measure urea, is commonly used but employs dangerous reagents.²⁴ Indirect methods measure ammonia created from the urea; the use of the enzyme urease has increased the specificity of these tests.²⁵ The ammonia is quantitated by a variety of methods, including nesslerization (acid titration), the Berthelot technique^{26,27} and coupled enzymatic reactions.^{28,29} Catalyzed Berthelot procedures, however, are erratic when measuring ammonia.³⁰ Coupled-enzyme reactions are rapid, have a high specificity for ammonia, and are commonly used. One such reaction has been proposed as a candidate reference method.³¹

In the coupled-enzyme reaction, urease hydrolyzes urea into ammonia and carbon dioxide. Upon combining ammonia with 2-oxoglutarate and reduced nicotinamide adenine dinucleotide (NADH), the enzyme glutamate dehydrogenase (GLDH) oxidizes NADH to NAD⁺.

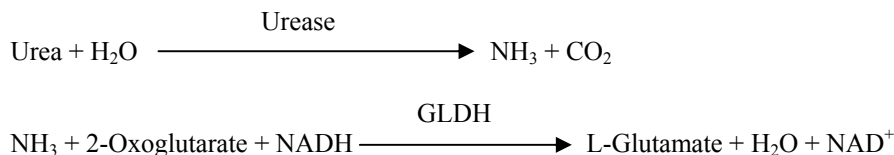

The rate of change of the absorbance difference between 340 nm and 405 nm is caused by the conversion of NADH to NAD⁺ and is directly proportional to the amount of urea present in the sample.

4. Principles of Procedure

See the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer Operator's Manual, for the Principles and Limitations of the Procedure.

5. Description of Reagents

Reagents

Each Piccolo General Chemistry 6 reagent disc contains dry test-specific reagent beads (described below). A dry sample blank reagent (comprised of buffer, surfactants, excipients, and preservatives) is included in each disc for use in calculating concentrations of alanine aminotransferase (ALT), aspartate aminotransferase (AST), gamma glutamyltransferase (GGT), glucose (GLU), and blood urea nitrogen (BUN). A dedicated sample blank is included in the disc for creatinine (CRE). Each reagent disc also contains a diluent consisting of surfactants, excipients, and preservatives.

Table 1: Reagents

Component	Quantity/Disc
Adenosine-5'-diphosphate	4 µg
Adenosine-5'-triphosphate	11 µg
L-Alanine	874 µg
4-Aminoantipyrine HCl	14 µg
Ascorbate oxidase (<i>Cucurbita</i> spp.)	0.4 U
L-Aspartic acid	426 µg
Creatine amidinohydrolase (<i>Actinobacillus</i> spp.)	2 U
Creatinine amidohydrolase (<i>Pseudomonas</i> spp.)	1 U
Glucose-6-phosphate dehydrogenase (yeast)	0.05 U
L-Glutamic acid dehydrogenase (bovine liver)	0.01 U
L-Glutamic acid γ-(3-carboxy-4-nitroanilide), ammonium salt	30 µg
Glycylglycine	317 µg
Hexokinase (yeast)	0.1 U
α-Ketoglutarate, disodium salt	28
α-Ketoglutaric acid	72 µg
Lactate dehydrogenase (chicken heart)	0.002 U
Lactate dehydrogenase (LDH) (microbial)	0.03 U
Lactate dehydrogenase (<i>Staphylococcus epidermidis</i>)	0.1 U
Magnesium acetate	7 µg
Malate dehydrogenase (MDH) (porcine heart)	0.01 U
Nicotinamide adenine dinucleotide (NAD ⁺)	20 µg
β-Nicotinamide adenine dinucleotide, reduced (NADH)	18 µg
Peroxidase (horseradish)	0.6 U
Potassium ferrocyanide	0.4 µg
Sarcosine oxidase (microorganism)	0.6 U
2,4,6-Tribromo-3-hydroxybenzoic acid	188 µg

Table 1: Reagents (continued)

Component	Quantity/Disc
Urease (jack bean)	0.05 U
Buffer, surfactants, excipients, and preservatives	

Warnings and Precautions

- For *In vitro* Diagnostic Use
- The diluent container in the reagent disc is automatically opened when the analyzer drawer closes. A disc with an opened diluent container cannot be re-used. Ensure that the sample or control has been placed into the disc before closing the drawer.
- Used reagent discs contain human body fluids. Follow good laboratory safety practices when handling and disposing of used discs.³² See the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer Operator's Manual for instructions on cleaning biohazardous spills.
- The reagent discs are plastic and may crack or chip if dropped. **Never** use a dropped disc as it may spray biohazardous material throughout the interior of the analyzer.
- Reagent beads may contain acids or caustic substances. The operator does not come into contact with the reagent beads when following the recommended procedures. In the event that the beads are handled (e.g., cleaning up after dropping and cracking a reagent disc), avoid ingestion, skin contact, or inhalation of the reagent beads.

Instructions for Reagent Handling

Reagent discs may be used directly from the refrigerator without warming. Do not allow discs to remain at room temperature longer than 48 hours prior to use. Open the sealed foil pouch and remove the disc, being careful not to touch the bar code ring located on the top of the disc. Use according to the instructions provided in the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer Operator's Manual. A disc not used within 20 minutes of opening the pouch should be discarded.

Storage

Store reagent discs in their sealed pouches at 2-8°C (36-46°F). Do not expose opened or unopened discs to direct sunlight or temperatures above 32°C (90°F). Reagent discs may be used until the expiration date included on the package. The expiration date is also encoded in the bar code printed on the bar code ring. An error message will appear on the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer display if the reagents have expired.

Indications of Reagent Disc Instability/Deterioration

A torn or otherwise damaged pouch may allow moisture to reach the unused rotor and adversely affect reagent performance. Do not use a rotor from a damaged pouch.

6. Instrument

See the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer Operator's Manual for complete information on use of the analyzer.

7. Sample Collection and Preparation

Sample collection techniques are described in the "Sample Collection" section of the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer Operator's Manual.

- The minimum required sample size is ~100 µL of heparinized whole blood, heparinized plasma, serum or control material. The reagent disc sample chamber can contain up to 120 µL of sample.
- Whole blood samples obtained by venipuncture must be homogeneous before transferring a sample to the reagent disc. Gently invert the collection tube several times just prior to sample transfer. Do not shake the collection tube; shaking may cause hemolysis.

- Whole blood venipuncture samples should be run within 60 minutes of collection.³³ **Glucose** concentrations are affected by the length of time since the patient has eaten and by the type of sample collected from the patient. To accurately determine glucose results, samples should be obtained from a patient who has been fasting for at least 12 hours. The glucose concentration decreases approximately 5-12 mg/dL in 1 hour in uncentrifuged samples stored at room temperature.³⁴
- Refrigerating whole blood samples can cause significant changes in concentrations of **aspartate aminotransferase, creatinine** and **glucose**.³⁵ The sample may be separated into plasma or serum and stored in capped sample tubes at 2-8°C (36-46°F) if the sample cannot be run within 60 minutes.
- Use only lithium heparin (green stopper) evacuated specimen collection tubes for whole blood or plasma samples. Use no-additive (red stopper) evacuated specimen collection tubes or serum separator tubes (red or red/black stopper) for serum samples.
- Start the test within 10 minutes of transferring the sample into the reagent disc.

8. Procedure

Materials Provided

- One Piccolo General Chemistry 6 Reagent Disc PN: 400-1006 (a box of discs PN: 400-0006)

Materials Required but not Provided

- Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer
- Sample transfer pipettes (fixed volume approximately 100 µL) and tips are provided with each Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer and may be reordered from Abaxis.
- Commercially available control reagents recommended by Abaxis (contact Abaxis Technical Service for approved control materials and expected values).
- Timer

Test Parameters

The Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer operates at ambient temperatures between 15°C and 32°C (59-90°F). The analysis time for each Piccolo General Chemistry 6 Reagent Disc is less than 14 minutes. The analyzer maintains the reagent disc at a temperature of 37°C (98.6°F) over the measurement interval.

Test Procedure

The complete sample collection and step-by-step operating procedures are detailed in the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer Operator's Manual.

Calibration

The Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer is calibrated by the manufacturer before shipment. The bar code printed on the bar code ring provides the analyzer with disc-specific calibration data. See the Piccolo Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer Operator's Manual.

Quality Control

See Section 2.4 of the Piccolo Operator's Manual or Section 6 (Calibration and Quality Control) of the Piccolo xpress Operator's Manual. Performance of the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer can be verified by running controls. For a list of approved quality control materials with acceptance ranges, please contact Abaxis Technical Support. Other human serum or plasma-based controls may not be compatible. Quality control materials should be stored as per the package-insert included with the controls.

If control results are out of range, repeat one time. If still out of range, call Technical Support. Do not report results if controls are outside their labeled limits. See the Piccolo or Piccolo xpress Operator's Manual for a detailed discussion on running, recording, interpreting, and plotting control results.

Waived Laboratories: Abaxis recommends control testing as follows:

- at least every 30 days

- whenever the laboratory conditions have changed significantly, e.g. Piccolo moved to a new location or changes in temperature control
- when training or retraining of personnel is indicated
- with each new lot (CLIA waived tests in waived status labs)

Non-Waived Laboratories: Abaxis recommends control testing to follow federal, state, and local guidelines.

9. Results

The Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer automatically calculates and prints the analyte concentrations in the sample. Details of the endpoint and rate reaction calculations are found in the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer Operator's Manual.

Interpretation of results is detailed in the Operator's Manual. Results are printed onto result cards supplied by Abaxis. The result cards have an adhesive backing for easy placement in the patient's files.

10. Limitations of Procedure

General procedural limitations are discussed in the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer Operator's Manual.

- The only anticoagulant **recommended for use** with the Piccolo Blood Chemistry System or the Piccolo xpress Chemistry System is **lithium heparin**. Do not use sodium heparin.
- Abaxis has performed studies demonstrating that EDTA, fluoride, oxalate, and any anticoagulant containing ammonium ions will interfere with at least one chemistry contained in the Piccolo General Chemistry 6 Reagent Disc.
- Samples with hematocrits in excess of 62-65% packed red cell volume (a volume fraction of 0.62-0.65) may give inaccurate results. Samples with high hematocrits may be reported as hemolyzed. These samples may be spun down to get plasma then re-run in a new reagent disc.
- **Any result for a particular test that exceeds the assay range should be analyzed by another approved test method or sent to a referral laboratory. Do not dilute the sample and run it again on the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer.**

Warning: Extensive testing of the Piccolo Blood Chemistry System or the Piccolo xpress Chemistry Analyzer has shown that, in very rare instances, sample dispensed into the reagent disc may not flow smoothly into the sample chamber. Due to the uneven flow, an inadequate quantity of sample may be analyzed and several results may fall outside the reference ranges. The sample may be re-run using a new reagent disc.

Interference

Substances were tested as interferents with the analytes. Human serum pools were prepared. The concentration at which each potential interferent was tested was based on the testing levels in NCCLS EP7-P.³⁶

Effects of Endogenous Substances

- Physiological interferents (hemolysis, icterus and lipemia) cause changes in the reported concentrations of some analytes. The sample indices are printed on the bottom of each result card to inform the operator about the levels of interferents present in each sample.
- The Piccolo Blood Chemistry System or the Piccolo xpress Chemistry Analyzer suppresses any results that are affected by >10% interference from hemolysis, lipemia or icterus. "HEM", "LIP", or "ICT" respectively, is printed on the result card in place of the result.
- For maximum levels of endogenous substances contact Abaxis Technical Service.

Effects of Exogenous and Therapeutic Substances

- Thirty-five exogenous and therapeutic substances were selected as potential interferents for Abaxis test methods based on recommendations by Young.³⁷ Significant interference is defined as a >10% shift in the result for a normal range

specimen. Human serum pools were supplemented with a known concentration of the drugs or chemicals and then analyzed.

Table 2: Exogenous & Therapeutic Substances Evaluated

	Physiologic or Therapeutic Range³⁶⁻⁴¹ (mg/dL)	Highest Concentration Tested (mg/dL)
Acetaminophen	1-2	100
Acetoacetate	0.05-3.60	102
Acetylsalicylic acid	2-10	50
Ampicillin	0.5	30
Ascorbic Acid	0.8-1.2	20
Caffeine	0.3-1.5	10
Calcium Chloride	—	20
Cephalothin (Keflin)	10	400
Chloramphenicol	1-2.5	100
Cimetidine	0.1-1	16
L-Dopa	—	5
Dopamine	—	19
Epinephrine	—	1
Erythromycin	0.2-2.0	10
Glutathione	—	30
Ibuprofen	0.5-4.2	50
Isoniazide	0.1-0.7	4
α -Ketoglutarate	—	5
Ketoprofen	—	50
Methicillin	—	100
Methotrexate	0.1	0.5
Metyldopa	0.1-0.5	0.5
Metronidazole	0.1	5
Nafcillin	—	1
Nitrofurantoin	0.2	20
Oxacillin	—	1
Oxaloacetate	—	132
Phenytoin	1-2	3
Proline	—	4
Pyruvate	0.3-0.9	44
Rifampin	0.4-3	1.5
Salicylic Acid	15-30	25
Sulfalazine	2-4	10
Sulfanilamide	10-15	50
Theophylline	1-2	20

- The following substances showed greater than 10% interference. Significant interference is defined as >10% shift in the result for a normal range specimen. Human serum pools were supplemented with known concentrations of the drugs or chemicals and then analyzed.

Table 3: Substances With Significant Interference >10%

	Physiologic/ Therapeutic Range³⁶⁻⁴¹ (mg/dL)	Concentration with > 10% Interference (mg/dL)	% Interference
Alanine Aminotransferase (ALT)			
Ascorbic acid	0.8-1.2	20	11% inc*
Oxaloacetate	—	132	843% inc
Creatinine (CRE)			
Ascorbic acid	0.8-1.2	20	11% dec
Dopamine	—	19	80% dec
L-dopa	—	5	71% dec
Epinephrine	—	1	45% dec
Glutathione	—	30	13% dec
Glucose (GLU)			
Oxaloacetate	—	132	11% dec
Pyruvate	0.3-0.9	44	13% dec

* inc=increase; dec=decrease.

For additional information on potential chemical interferents, see the Bibliography.

11. Expected Values

Samples from a total of 193 adult males and females, analyzed on the Piccolo Blood Chemistry Analyzer, were used to determine the reference ranges for ALT, creatinine, glucose, and BUN. Samples from a total of 186 adult males and females were used to determine the reference range for AST. Samples from a total of 131 adult males and females were used to determine the reference range for GGT. These ranges are provided as a guideline only. It is recommended that your office or institution establish normal ranges for your particular patient population.

Table 4: Piccolo Reference Intervals

Analyte	Common Units	SI Units
Alanine Aminotransferase (ALT)	10-47 U/L	10-47 U/L
Aspartate Aminotransferase (AST)	11-38 U/L	11-38 U/L
Creatinine (CRE)	0.6-1.2 mg/dL	53-106 µmol/L
Gamma Glutamyltransferase (GGT)	5-65 U/L	5-65 U/L
Glucose (GLU)	73-118 mg/dL	4.05-6.55 mmol/L
Blood Urea Nitrogen (BUN)	7-22 mg/dL	2.5-7.9 mmol urea/L

12. Performance Characteristics

Linearity

The chemistry for each analyte is linear over the dynamic range listed below when the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer is operated according to the recommended procedure (refer to the Piccolo Blood Chemistry Analyzer or the Piccolo xpress Chemistry Analyzer Operator's Manual).

Table 5: Piccolo Dynamic Ranges

Analyte	Common Units	SI Units
Alanine Aminotransferase (ALT)	5-2000 U/L	5-2000 U/L
Aspartate Aminotransferase (AST)	5-2000 U/L	5-2000 U/L
Creatinine (CRE)	0.2-20 mg/dL	18-1768 µmol/L
Gamma Glutamyltransferase (GGT)	5-3000 U/L	5-3000 U/L
Glucose (GLU)	10-700 mg/dL	0.56-38.9 mmol/L
Blood Urea Nitrogen (BUN)	2-180 mg/dL	0.7-64.3 mmol/urea/L

If the analyte concentration is above the measuring range (dynamic range), but less than the system range, the print card will indicate a “>” sign at the upper limit and an asterisk after the number, e.g. ALT >2000* U/L. If lower than the dynamic range, a “<” will be printed with an asterisk, e.g. ALT <5* U/L. For values that are grossly beyond the measurement range (system range), “~~~~” will be printed instead of a result. Any time “~~~~” appears on a print card, collect a new sample and rerun the test. If results for the second sample are suppressed again, please call Abaxis Customer Support.

Sensitivity (Limits of Detection)

The lower limit of the reportable (dynamic) range for each analyte is: alanine aminotransferase 5 U/L; aspartate aminotransferase 5 U/L; creatinine 0.2 mg/dL (18 µmol/L); gamma glutamyltransferase 5 U/L; glucose 10 mg/dL (0.56 mmol/L); and blood urea nitrogen 2.0 mg/dL (0.7 mmol urea/L).

Precision

Precision studies were conducted using NCCLS EP5-T2 guidelines.⁴² Results for within-run and total precision were determined by testing two levels of control material. Controls were run in duplicate twice each day for 20 days over a four-week period. Results of the precision studies are shown in Table 6.

Table 6: Precision (N=80)

Analyte	Within-Run	Total
Alanine Aminotranferase (U/L)		
<u>Control Level 1</u>		
Mean	21	21
SD	2.76	2.79
%CV	13.4	13.5
<u>Control Level 2</u>		
Mean	52	52
SD	2.70	3.25
%CV	5.2	6.2
Aspartate Aminotransferase (U/L)		
<u>Control Level 1</u>		
Mean	47	49
SD	0.98	0.92
%CV	2.1	1.9
<u>Control Level 2</u>		
Mean	145	147
SD	1.83	1.70
%CV	1.3	1.2
Creatinine (mg/dL)		
<u>Control Level 1</u>		
Mean	1.1	1.1
SD	0.14	0.14
%CV	12.5	13.1

Table 6 Precision (N=80) (cont.)

Analyte	Within-Run	Total
<u>Control Level 2</u>		
Mean	5.2	5.2
SD	0.23	0.27
%CV	4.4	5.2
Gamma Glutamyltransferase (U/L)		
<u>Control Level 1</u>		
Mean	25	25
SD	0.59	0.74
%CV	2.34	2.94
<u>Control Level 2</u>		
Mean	106	106
SD	1.52	2.29
%CV	1.43	2.15
Glucose (mg/dL)		
<u>Control Level 1</u>		
Mean	66	66
SD	0.76	1.03
%CV	1.1	1.6
<u>Control Level 2</u>		
Mean	278	278
SD	2.47	3.84
%CV	0.9	1.4
Blood Urea Nitrogen (mg/dL)		
<u>Control Level 1</u>		
Mean	19	19
SD	0.35	0.40
%CV	1.9	2.1
<u>Control Level 2</u>		
Mean	65	65
SD	1.06	1.18
%CV	1.6	1.8

Correlation

Heparinized whole blood and serum samples were collected from patients at two sites. The whole blood samples were analyzed by the Piccolo Blood Chemistry Analyzer at the field sites and the serum samples were analyzed by comparative methods. In two cases, the results of testing serum samples by the Piccolo were used and these are indicated appropriately in the table. In some cases, high and low supplemented samples were used to cover the dynamic range. All samples were run in singlicate on the same day. Representative correlation statistics are shown in Table 7.

Table 7: Correlation of Piccolo Blood Chemistry Analyzer with Comparative Methods

	Correlation Coefficient	Slope	Intercept	SEE	N	Sample Range	Comparative Method
Alanine Aminotransferase (U/L)	0.981	0.905	1.3	3.21	86	10-174	Paramax® Technicon
	0.985	0.946	-2.5	2.84	67	10-174	
Aspartate Aminotransferase (U/L)	0.93	0.87	5.3	2.76	159	13-111	Paramax DAX™
	1.0	0.97	3.0	1.9	46	13-252	
Creatinine (mg/dL)	0.993	0.926	0.0	0.15	260	0.4-14.7	Paramax Beckman
	0.987	0.866	0.1	0.16	107	0.4-7.5	
Gamma Glutamyl-transferase (U/L)	1.0	0.98	-0.4	3.29	135	5-312	Paramax Beckman
	1.0*	1.60	3.1	18.57	49	27-1848	
Glucose (mg/dL)	0.987	1.009	-2.8	3.89	251	72-422	Paramax Beckman
	0.997	0.943	1.2	4.69	91	56-646	
Urea Nitrogen (mg/dL)	0.964	0.923	0.5	1.08	251	6 –52	Paramax Beckman
	0.983	0.946	0.0	0.66	92	6-38	

* One site ran only serum on the Piccolo analyzer for the gamma glutamyltransferase correlation.

Results of Untrained User Study

An “untrained user” study was conducted in which participants were given only the test instructions and asked to perform testing of 3 discs with blinded randomized samples. The samples consisted of serum pools prepared at three levels for each of the thirteen analytes: ALT, AST, creatinine, GGT, glucose, and BUN. The participants were not given any training on the use of the test. A total of approximately 60 participants were enrolled from 3 sites, representing a diverse demographic (educational, age, gender, etc) population.

Tables below present the summary of the performance for each analyte.

Alanine Aminotransferase (ALT)

	Level 1	Level 2	Level 3
N	62	62	62
Mean	45.4 U/L	98.9 U/L	184.3 U/L
%CV	3.7%	1.7%	1.5%
Observed Range	42 – 53	96 – 103	175 – 191
Percent of Results in the Range ± 15.0%*	98.4% 61/62 95%CI: 91.3% to 100%	100% 62/62 95%CI: 94.2% to 100%	100% 62/62 95%CI: 94.2% to 100%

* This percent is based on the premise that one cannot distinguish properly between normal and abnormal values when errors are greater than one-quarter of the normal range. The range of (10 U/L - 47 U/L) was considered.

Aspartate Aminotransferase (AST)

	Level 1	Level 2	Level 3
N	62	62	62
Mean	56.0	120.4	276.3
%CV	2.4%	1.1%	1.0%
Observed Range	54 – 60	117 – 124	266 – 285
Percent of Results in the Range ± 15.0%	100% 62/62 95%CI: 94.2% to 100%	100% 62/62 95%CI: 94.2% to 100%	100% 62/62 95%CI: 94.2% to 100%

Creatinine (CRE)

	Level 1	Level 2	Level 3
N	62	62	62
Mean	0.89	2.07	6.89
%CV	11.0	5.0	1.6
Observed Range	0.7 – 1.2	1.8 – 2.3	6.5 – 7.2
Percent of Results in the Range ± 15.0%	93.6 58/62 95%CI: 84.3% to 98.2%	100% 62/62 95%CI: 94.2% to 100%	100% 62/62 95%CI: 94.2% to 100%

Gamma Glutamyltransferase (GGT)

	Level 1	Level 2	Level 3
N	62	62	62
Mean	35.0 U/L	86.2 U/L	131.3 U/L
%CV	2.8%	1.5%	1.5%
Observed Range	33 – 38	83 – 90	123 – 135
Percent of Results in the Range ± 15.0%	100% 62/62 95%CI: 94.2% to 100%	100% 62/62 95%CI: 94.2% to 100%	100% 62/62 95%CI: 94.2% to 100%

Glucose (GLU)

	Level 1	Level 2	Level 3
N	62	62	62
Mean	95.2	130.3	365.8
%CV	1.1%	1.0%	0.8%
Observed Range	93 – 98	125 – 133	351 – 373
Percent of Results in the Range ± 10.4%**	100% 62/62 95%CI: 94.2% to 100%	100% 62/62 95%CI: 94.2% to 100%	100% 62/62 95%CI: 94.2% to 100%

** The range of (65 mg/dL - 99 mg/dL) was considered.

Blood Urea Nitrogen (BUN)

	Level 1	Level 2	Level 3
N	62	62	62
Mean	15.1	41.0	72.2
%CV	2.3	2.5	1.8
Observed Range	14 – 16	37 – 43	68 – 75
Percent of Results in the Range ± 15.0%	100% 62/62 95%CI: 94.2% to 100%	100% 62/62 95%CI: 94.2% to 100%	100% 62/62 95%CI: 94.2% to 100%

13. Bibliography

1. Tonhazy NE, NG White, WW Umbreit. A rapid method for the estimation of the glutamic-aspartic transaminase in tissues and its application to radiation sickness. Arch Biochem 1950; 28: 36-42.
2. Reitman S, Frankel S. A colorimetric method for the determination of serum glutamic oxalacetic and glutamic pyruvic transaminases. Am J Clin Pathol 1957; 28: 56-63.
3. Murray RL. Alanine aminotransferase. In: Clinical Chemistry: Theory, Analysis, and Correlation, 2nd ed. Kaplan LA, Pesce AJ, eds. St. Louis: The C.V. Mosby Company. 1989: 895-898.
4. Wróblewski F, LaDue JS. Serum glutamic-pyruvic transaminase in cardiac and hepatic disease. Proc Soc Exp Biol Med 1956; 91: 569-571.
5. Bergmeyer HU, Horder M. IFCC methods for the measurement of catalytic concentration of enzymes. Part 3. IFCC method for alanine aminotransferase. J Clin Chem Clin Biochem 1980; 18: 521-534.
6. Karmen A. A note on the spectrophotometric assay of glutamic-oxalacetic transaminase in human blood serum. J Clin Invest 1955; 34: 131-133.

13. Bibliography (cont.)

7. Bergmeyer, HU, et al. Provisional recommendations on IFCC methods for the measurement of catalytic concentrations of enzymes. Part 2. IFCC method for aspartate aminotransferase. *Clin Chem* 1977; 23: 887-899.
8. Bergmeyer HU, Hørdler M, Moss DW. Provisional recommendations on IFCC methods for the measurement of catalytic concentrations of enzymes. Part 2. Revised IFCC method for aspartate aminotransferase. *Clin Chem* 1978; 24: 720-721.
9. Knoll VE, Stamm D. Spezifische kreatininbestimmung im serum. *Z Klin Chem Klin Biochem* 1970; 8: 582-587.
10. Haeckel R. Simplified determinations of the "true" creatinine concentration in serum and urine. *J Clin Chem Clin Biochem* 1980; 18: 385-394.
11. Moss GA, Bondar RJL, Buzzelli DM. Kinetic enzymatic method for determining serum creatinine. *Clin Chem* 1975; 21: 1422-1426.
12. Jaynes PK, Feld RD, Johnson GF. An enzymic, reaction-rate assay for serum creatinine with a centrifugal analyzer. *Clin Chem* 1982; 28: 114-117.
13. Fossati P, Prencipe L, and Berti G. Enzymic creatinine assay: a new colorimetric method based on hydrogen peroxide measurement. *Clin Chem* 1983; 29: 1494-1496.
14. Whelton A, Watson AJ, Rock RC. Nitrogen metabolites and renal function. In: Tietz Textbook of Clinical Chemistry, 2nd ed. Burtis CA, Ashwood ER, eds. Philadelphia: W.B. Saunders Company. 1994: 1513-1575.
15. Ball, EG, Revel JP, Cooper O. The quantitative measurement of γ -glutamyl transpeptidase activity. *J Biol Chem* 1956; 221: 895-908.
16. Goldbarg JA, et al. The colorimetric determination of γ -glutamyl transpeptidase with a synthetic substrate. *Arch Biochem Biophys* 1960; 91: 61-70.
17. Orłowski M, Meister A. γ -Glutamyl-p-nitroanilide: a new convenient substrate for determination and study of L- and D γ -glutamyltranspeptidase activities. *Biochim Biophys Acta* 1963; 73: 679-681.
18. Persijn JP, van der Slik W. A new method for the determination of γ -glutamyltransferase in serum. *J Clin Chem Clin Biochem* 1976; 14: 421-427.
19. Shaw LM, et al. IFCC methods for the measurement of catalytic concentration of enzymes. Part 4. IFCC method for γ -glutamyltransferase. *J Clin Chem Clin Biochem* 1983; 21: 633-646.
20. Folin O, Wu H. A system of blood analysis. *J Biol Chem* 1919; 38: 81-110.
21. Somogyi M. A reagent for the copper-iodometric determination of very small amounts of sugar. *J Biol Chem* 1937; 117: 771-776.
22. Nelson N. A photometric adaptation of the Somogyi method for the determination of glucose. *J Biol Chem* 1944; 153: 375-380.
23. Kaplan LA. Glucose. In: *Clinical Chemistry: Theory, Analysis, and Correlation*, 2nd ed. Kaplan LA, Pesce AJ, eds., St. Louis: The C.V. Mosby Company. 1989: 850-856.
24. Fales FW. Urea in serum, direct diacetyl monoxime method. In: *Selected Methods of Clinical Chemistry*, vol 9. Faulkner WR, Meites S, eds. Washington, D.C.: American Association for Clinical Chemistry. 1982: 365-373.
25. Van Slyke DD, Cullen GE. A permanent preparation of urease, and its use in the determination of urea. *J Biol Chem* 1914; 19: 211-228.
26. Fawcett JK, Scott JE. A rapid and precise method for the determination of urea. *J Clin Pathol* 1960; 13: 156-159.
27. Chaney AL, Marbach EP. Urea and ammonia determinations. *Clin Chem* 1962; 8: 130-132.
28. Talke H, Schubert GE. Enzymatische Harnstoffbestimmung in Blut und Serum im optischen Test nach Warburg. *Klin Wochensh* 1965; 43: 174-175.
29. Hallett CJ, Cook JGH. Reduced nicotinamide adenine dinucleotide-coupled reaction for emergency blood urea estimation. *Clin Chim Acta* 1971; 35: 33-37.
30. Patton CJ, Crouch SR. Spectrophotometric and kinetics investigation of the Berthelot reaction for the determination of ammonia. *Anal Chem* 1977; 49: 464-469.
31. Sampson EJ, et al. A coupled-enzyme equilibrium method for measuring urea in serum: optimization and evaluation of the AACC study group on urea candidate reference method. *Clin Chem* 1980; 26: 816-826.
32. National Committee for Clinical Laboratory Standards. Physician's office laboratory guidelines; tentative guideline – second edition. NCCLS Document POL1-T2 Wayne, PA: NCCLS, 1992.
33. National Committee for Clinical Laboratory Standards. Procedures for the handling and processing of blood specimens; approved guideline – second edition. NCCLS document H18-A2. Wayne, PA: NCCLS, 1999.
34. Overfield CV, Savory J, Heintges MG. Glycolysis: a re-evaluation of the effect on blood glucose. *Clin Chim Acta* 1972; 39: 35-40.
35. Rehak NN, Chiang BT. Storage of whole blood: effect of temperature on the measured concentration of analytes in serum. *Clin Chem* 1988; 34: 2111-2114.
36. National Committee for Clinical Laboratory Standards. Interference testing in clinical chemistry; proposed guideline. NCCLS Publication EP7-P. Wayne, PA: NCCLS, 1986.
37. Young DS. *Effects of drugs on clinical laboratory tests*, 3rd ed. Washington, DC: AACC Press. 1990.

13. Bibliography (cont.)

38. Benet LZ, Williams RL. Design and optimization of dosage regimens: pharmacokinetic data. In: Goodman and Gilman's The Pharmacological Basis of Therapeutics, 8th ed. Gilman AG, et al, eds. New York: McGraw-Hill, Inc. 1990: 1650-1735.
39. Young DS. Effects of drugs on clinical laboratory tests. 1991 supplement to the third edition. Washington, DC: AACC Press. 1991.
40. Moss DW, Henderson AR. Enzymes. In: Tietz Textbook of Clinical Chemistry, 2nd ed. Burtis CA, Ashwood ER, eds. Philadelphia: W.B. Saunders Company. 1994: 735-896.
41. Painter PC, Cope JY, Smith JL. Appendix. In: Tietz Textbook of Clinical Chemistry, 2nd ed. Burtis CA, Ashwood ER, eds. Philadelphia: W.B. Saunders Company. 1994: 2161-2217.
42. National Committee for Clinical Laboratory Standards. Evaluation of precision performance of clinical chemistry devices; tentative guideline – second edition. NCCLS Document EP5-T2. Wayne, PA: NCCLS, 1992.