



Accurate. Easy. Affordable.

The VetScan Canine Heartworm Rapid Test is a highly sensitive and specific test for the detection of *Dirofilaria immitis* in canine whole blood, serum or plasma. The VetScan Canine Heartworm Rapid Test delivers speed, simplicity and VetScan quality for a price significantly lower than other brands.

- Simple, Easy to Use
- Fast—Results in Minutes
- Easy-to-Read Results
- Uses Whole Blood, Serum or Plasma
- Room Temperature Storage
- 12-Month Shelf Life
- Available in 25-Test Kit or 100-Test Kit



Clinically Validated Confidence

The VetScan Canine Heartworm Rapid Test delivers quality that is equal to or better than other in-office lateral flow devices. *D. immitis* status was determined by necropsy from experimentally infected dogs (disease history determined).

Sensitivity	VetScan	Snap®	Solo Step®	Witness®
1 female worm only: n=22	21 (95%)	21 (95%)	21 (95%)	21 (95%)
2 female worms only: n=25	25 (100%)	25 (100%)	25 (100%)	24 (96%)
3+ female worms: n=10	10 (100%)	10 (100%)	10 (100%)	10 (100%)
Overall Sensitivity	VetScan	Snap	Solo Step	Witness
n=57	98%	98%	98%	96%
Overall Specificity	VetScan	Snap	Solo Step	Witness
n=50	100%	100%	100%	100%

All testing performed at TRS Laboratories, Athens, GA. John McCall, PhD, University of Georgia at Athens.



www.abaxis.com

Abaxis North America

3240 Whipple Road
Union City, CA 94587
Tel 800 822 2947
Fax 510 441 6150

Abaxis Europe

Otto-Hesse-Strasse 19
T9, 3. OG Ost
D-64293 Darmstadt Germany
Tel +49 6151 350 790
Fax +49 6151 350 7911